

IF YOU HAVE TO REUSE

Use Low **DEAD SPACE**



FOR

*Easy Cleaning*

# Rinsing

IS MORE EFFECTIVE  
WITH LOW DEAD SPACE  
NEEDLES & SYRINGES



BE WISE

# STERILISE


Fill two clean cups with cold water and one with a capful of thin bleach

1

Draw up water



Flush

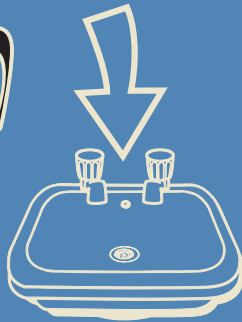


2

Draw up thin bleach



Flush

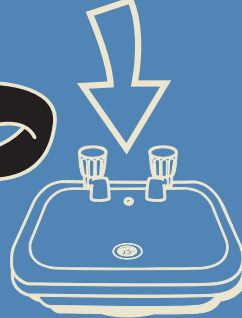


3

Draw up water



Flush

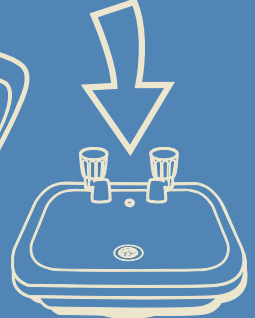

One rinse removes the bleach

EASY  
**CLEAN**

USE

LESS  
**RISK**

**LOW DEAD SPACE**  
NEEDLES & SYRINGES